


1. Remove the frame of the Centri Clean unscrewing 3 fixation screws with a Phillips Screwdriver.



2. Once removed the screw, remove the frame.



3. Remove the screw that fixes the latch.

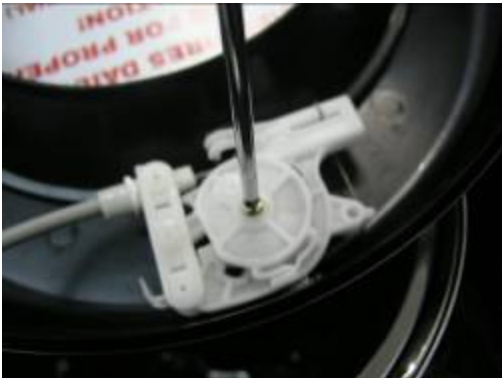

4. Use a small screwdriver as a lever to remove the round white plastic device of the latch.



5. Remove the other white plastic part of the latch by carefully pulling it up.



6. There is a small plastic piece that fixes the whole thing together. Detach, carefully, one side from the other by pulling both pieces in different ways. Then remove all pieces from it.





7. Remove the spring by pulling up one part at a time.



8. Turn the metal piece so the tip can pass through the fitting of the white plastic piece.



9. Remove the plastic piece of the cable from its fitting place.

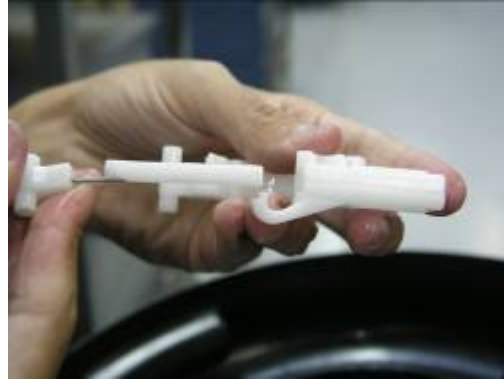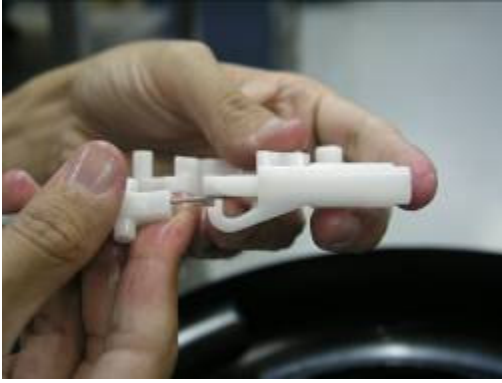

To assemble the latch, follow the steps in reverse.